WARNINGS

- The user must be fully acquainted with surgical procedures and techniques.
- Select the appropriate type of knife for each surgical purpose. Use in accordance with normal surgical procedures.
- Do not use the knife if its packaging is damaged or if there is evidence of any other abnormalities.

CONTRAINDICATIONS AND PROHIBITIONS

- Do not use on patients who are sensitive or allergic to metals.
- Do not use for any other purpose than ophthalmic surgery.
- Do not use if the blade has been hit against or been in contact with anything other than the eye, as the blade's sharpness might be compromised.
- Do not reuse or re-sterilize. Reuse or re-sterilization could seriously compromise the structural integrity and / or might lead to damage or infection to patients.

Materials - shape and structure

The product is a sterile knife with handle.

Handle: Polycarbonate or Polysulfone

Blade: 400 Series Stainless steel

INTENDED USE

Indications: Labtician Ophthalmic Knives are used in the surgical treatment of various diseases of the eye. E.g., cataract, retinal, glaucoma and other procedures to create an incision in the eye. As such, they are only to be used by licensed, trained physicians. The specific nature of the disease will determine the physician's choice of product and the techniques used.

Microsurgical knife is used in microsurgical procedures for cutting. Supplied sterile and is disposable.

PERFORMANCE, PURPOSE AND EFFECT

To make surgical incisions.

OPERATION AND USE

- Use in accordance with ordinary incision procedures.

CAUTION

- Use the knife as soon as it is unpacked.
- Do not use deformed or fractured knives.
- Do not disassemble or distort the shape of the knife before use.
- This knife has a very sharp blade. Take extreme care when handling to avoid unintentional sticks to healthcare providers or patients. External injury caused by a contaminated blade can lead to pathogenic infection linked to blood disorders. If unintentional stick occurs, do not use the device.
- To prevent injury or damage, avoid contact between the blade edge and any surfaces prior to use.

- Do not use excessive force, or use with inappropriate equipment.
- Sterility is guaranteed unless opened or damaged. Do not use if the sterile pack has been opened or damaged.
- Do not re-sterilize or reuse. Labtician Ophthalmic, Inc. assumes no liability for devices that have been altered, dismantled, reassembled, re-sterilized and/or reused.
- Please use Labtician Ophthalmic Surgical Knives and other instruments responsibly. To prevent infection, dispose of with care in a safe and environmentally friendly manner, such as in a medical waste basket.
- Federal (USA) law restricts this device to sale by, or on the order of a physician.
- Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

METHOD OF STERILIZATION **STERILE R**

Labtician's ophthalmic knives are gamma sterilized to SAL 10⁻⁶ and must be opened under aseptic conditions.

CONDITIONS OF STORAGE

Labtician knives should be stored at ambient temperature away from possible contamination from biohazards and water damage.

EXPIRATION DATE

The expiration date is clearly indicated on the packaging. Any ophthalmic knives held after the expiration date should not be used.

INSTRUCTIONS

1. Select the style and size of Labtician ophthalmic surgical knife indicated for the patient and procedure to be performed.
2. Remove product from sterile packaging carefully. Labtician knives are surgically sharp. Care should be taken when removing knives from packaging, in handling, and in disposal.
3. Physician to perform procedure as necessary taking precautions to avoid accidental cuts to self, medical personnel and patient.
4. After use, discard the knife into an approved sharps container.

	Lot number		Product is classified as a Medical Device
	Sterilized using gamma		Product Code
	Do not use if packaging is damaged		Unique Device Identifier
	Do not reuse		Use by date
	Consult instructions for Use		Do not resterilize
www.Labtician.com/docs			Keep Dry
	Caution		Prescription Only
	Single sterile barrier system		

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