IMPORTANT NOTICE

It is highly recommended that the surgeon adhere to the cautions outlined in these instructions.

DEVICE DESCRIPTION

Labtician's non-vacuum trephine punches are standard ophthalmic devices consisting of a stainless steel blade, a plastic holder and four stainless steel guide pins. They are delivered sterile to the end user and are intended for single patient use.

Materials

- Blade: stainless steel
- Holder: plastic
- Guide Pins: stainless steel

INTENDED USE

The Labtician non-vacuum trephine punch is a disposable system designed for use in Corneal Transplant Surgery, particularly for preparing the donor cornea.

CAUTION

- The trephine blade is exceptionally sharp. Avoid contact between the blade edge and any surfaces prior to use to avoid injury or damage.
- Care should be taken at all times to avoid unintentional sticks to healthcare providers or patient. If unintentional stick occurs, do not use the device.
- Do not use excessive force, or use with inappropriate equipment.
- Labtician's non-vacuum trephine punches are sold sterile. Sterility is guaranteed unless opened or damaged. Do not use if the sterile pack has been opened or damaged.
- Do not re-sterilize or reuse. Labtician assumes no liability for devices that have been altered, dismantled, reassembled, re-sterilized or reused.
- Please use Labtician's non-vacuum trephine punches responsibly and remember to dispose of in a safe and environmentally friendly manner.
- Federal (USA) law restricts this device to sale by, or on the order of a physician.

METHOD OF STERILIZATION STERILE R

Labtician's non-vacuum trephine punches are gamma sterilized to SAL 10⁻⁶ and must be opened under aseptic conditions.

CONDITIONS OF STORAGE

Labtician's non-vacuum trephine punches should be stored in a cool (60°F – 70°F), dry place, away from sources of heat or sunlight, and possible contamination from biohazards and water damage.

EXPIRATION DATE

The expiration date is clearly indicated on the packaging. Any non-vacuum trephine punch held after the expiration date should not be used.

RETURN GOODS POLICY

Labtician accepts returned non-vacuum trephine punches for exchanges or account credit only. No cash refunds will be issued. To return these devices, you must first obtain a return authorization number from our customer service department. No returned goods will be accepted without proper authorization number. Returned goods should

be shipped by traceable method. No credit will be given for lost or damaged devices in shipment. Devices will be replaced or your account credited as long as they are returned within six months of their original invoice date.

PREPARATION

Remove unit from box and verify the blade diameter indicated on the top of the punch. The unit consists of two pieces - the trephine portion (top) and the well portion (bottom). Remove the packaging band that holds the two pieces of the unit together. Pull trephine portion straight and evenly off the unit and place both pieces on procedural field with the flat side down. Discard the protective ring surrounding the trephine and the packaging band.

MARKING THE DONOR CORNEA

In order to mark the four quadrants on the epithelial side of the donor cornea, place the tip of a sterile surgical marking pen (gentian violet) into the four holes located in the well portion. Evenly coat each hole by twisting the pen.

CUTTING THE DONOR CORNEA

Place donor cornea in the well portion with the endothelial side facing up. Cut away all sclera tissue extending beyond the well portion. Center the tissue with fine tooth forceps before cutting. Next, insert the four pins of the trephine portion into the four holes located in the corners of the well portion. Press down firmly on the flat surface of the trephine portion using your thumb.

REMOVAL OF DONOR CORNEA

Pull top of trephine punch straight and evenly off the tissue.

The Donor button typically will remain in the well portion, while the peripheral tissue will pull away with the trephine portion.

Should the button pull away with the trephine portion and not remain in the well portion, it may be dislodged with Balanced Salt Solution or Viscoelastic from the rear (endothelial) side, which is through the center of trephine.

	Lot number		Product is classified as a Medical Device
	Sterilized using gamma		Product Code
	Do not use if packaging is damaged		Unique Device Identifier
	Do not reuse		Use by date
	Consult instructions for Use		Do not resterilize
www.Labtician.com/docs			Keep Dry
	Caution		Prescription Only
	Single sterile barrier system		

INTENDED USE:

The Labtician Vacuum Trephine is a disposable system designed for use in Corneal Transplant Surgery.

PRODUCT DETAILS AND CAUTIONS:

- The unit is supplied sterile and ready to use for single use only. Do not re-sterilize or reuse.
- Carefully inspect the packaging. Do not use if the package has been opened or damaged.
- After opening the package, please exercise caution at all times when handling and using the trephine.
- The trephine blade is exceptionally sharp. Avoid contact between the blade edge and any surfaces prior to use, to avoid injury or damage.
- Labtician assumes no liability for devices that have been altered, dismantled, reassembled, re-sterilized and/or reused.

USING THE SYSTEM (see Figure 1)

Under the microscope, align the trephine cutting edge with that of the inner tube by rotating the ring. For future alignment reference, take note of the position of the LONG marker pin on the ring.

From this position, make two complete counterclockwise rotations, recessing the trephine deeper into the inner tube, thus providing a more effective vacuum (see Figure 2).

Looking through the lumen and centering the crosshairs, carefully position and press the unit down on the cornea.

Fully depress the suction syringe plunger and hold.

Release and wait about 10 seconds for adequate suction to develop. Test suction by gently lifting the unit. Then permit the cornea to return to its normal shape before the next step.

Advance the trephine blade forward by rotating the ring clockwise two full turns. This once again aligns the trephine edge with the edge of the inner tube.

Cutting is now achieved via clockwise rotation of the ring. Each quarter-rotation advances the cutting surface downward by .063 mm (for a total of .25 mm per one complete rotation) (see Figure 3).

Rotate slowly and carefully, using the pins for greater control.

Discontinue cutting at the first sign of aqueous.


To release the suction, either depress the plunger or cut the tubing.

THESE INSTRUCTIONS APPLY TO THE FOLLOWING DEVICES:



REF.	Description
LAB350600	6.00 mm Vacuum Trephine
LAB350625	6.25 mm Vacuum Trephine
LAB350650	6.50 mm Vacuum Trephine
LAB350675	6.75 mm Vacuum Trephine
LAB350700	7.00 mm Vacuum Trephine
LAB350725	7.25 mm Vacuum Trephine
LAB350750	7.50 mm Vacuum Trephine
LAB350775	7.75 mm Vacuum Trephine
LAB350785	7.85 mm Vacuum Trephine
LAB350800	8.00 mm Vacuum Trephine
LAB350825	8.25 mm Vacuum Trephine
LAB350850	8.50 mm Vacuum Trephine
LAB350875	8.75 mm Vacuum Trephine
LAB350900	9.00 mm Vacuum Trephine
LAB350925	9.25 mm Vacuum Trephine
LAB350950	9.50 mm Vacuum Trephine
LAB351000	10.00 mm Vacuum Trephine
LAB370600	6.00 mm Advanced Radial Vacuum Trephine
LAB370650	6.50 mm Advanced Radial Vacuum Trephine
LAB370700	7.00 mm Advanced Radial Vacuum Trephine
LAB370725	7.25 mm Advanced Radial Vacuum Trephine
LAB370750	7.50 mm Advanced Radial Vacuum Trephine
LAB370775	7.75 mm Advanced Radial Vacuum Trephine
LAB370800	8.00 mm Advanced Radial Vacuum Trephine
LAB370825	8.25 mm Advanced Radial Vacuum Trephine
LAB370850	8.50 mm Advanced Radial Vacuum Trephine
LAB370875	8.75 mm Advanced Radial Vacuum Trephine
LAB370900	9.00 mm Advanced Radial Vacuum Trephine

FIGURE 1

Note: One pin is larger than others for orientation

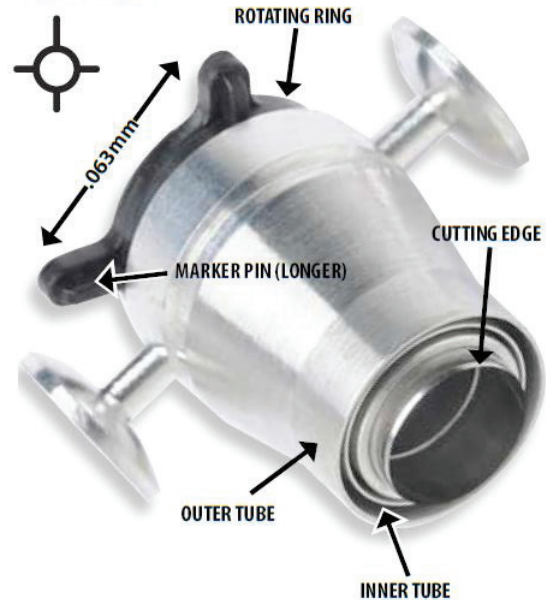Picture shows cutting edge advanced forward

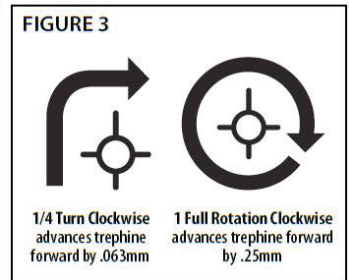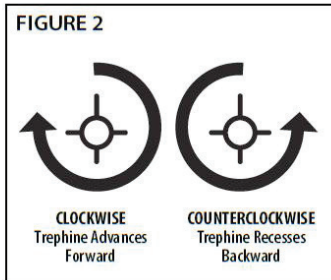	Lot number		Product is classified as a Medical Device
	Sterilized using gamma		Product Code
	Do not use if packaging is damaged		Unique Device Identifier
	Do not reuse		Use by date
	Consult instructions for Use <small>www.Labtician.com/docs</small>		Do not resterilize
	Caution		Keep Dry
	Single sterile barrier system		Prescription Only

IMPORTANT NOTICE

It is highly recommended that the surgeon adhere to the cautions outlined in these instructions.

DEVICE DESCRIPTION

Labtician's vacuum trephine punches are standard ophthalmic devices consisting of a stainless steel blade, a plastic holder, a silicone tube and a handheld polypropylene syringe with a stainless steel spring used to create a vacuum during corneal procedures. They are delivered sterile to the end user and are intended for single patient use.

Materials

- Blade: stainless steel
- Holder: plastic
- Tube: medical grade silicone
- Syringe: polypropylene
- Spring: stainless steel

INTENDED USE

The Labtician vacuum trephine punch is a disposable system designed for use in Corneal Transplant Surgery, particularly for preparing the donor cornea.

CAUTION

- Care should be taken at all times to avoid unintentional
- The trephine blade is exceptionally sharp. Avoid contact between the blade edge and any surfaces prior to use to avoid injury or damage.
- Care should be taken at all times to avoid unintentional sticks to healthcare providers or patient. If unintentional stick occurs, do not use the device.
- Do not use excessive force, or use with inappropriate equipment.
- Labtician's vacuum trephine punches are sold sterile. Sterility is guaranteed unless opened or damaged. Do not use if the sterile pack has been opened or damaged.
- Do not re-sterilize or reuse. Labtician assumes no liability for devices that have been altered, dismantled, reassembled, re-sterilized or reused.
- Please use Labtician's vacuum trephine punches responsibly and remember to dispose of in a safe and environmentally friendly manner.
- Federal (USA) law restricts this device to sale by, or on the order of a physician.

METHOD OF STERILIZATION STERILE R

Labtician's vacuum trephine punches are gamma sterilized to SAL 10⁻⁶ and must be opened under aseptic conditions.

CONDITIONS OF STORAGE

Labtician's vacuum trephine punches should be stored in a cool (60°F – 70°F), dry place, away from sources of heat or sunlight, and possible contamination from biohazards and water damage.

EXPIRATION DATE

The expiration date is clearly indicated on the packaging. Any vacuum trephine punch held after the expiration date should not be used.

RETURN GOODS POLICY

Labtician accepts returned vacuum trephine punches for exchanges or account credit only. No cash refunds will be issued. To return these devices, you must first obtain a return authorization number from our customer service department. No returned goods will be accepted without proper authorization number. Returned goods should be shipped by traceable method. No credit will be given for lost or damaged devices in shipment. Devices will be replaced or your account credited as long as they are returned within six months of their original invoice date.

PREPARATION

Remove unit from box and verify the blade diameter indicated on the top of the punch. The unit consists of three pieces - the trephine portion (top), the well portion with suction syringe (bottom) and the seating ring. Remove the packaging band that holds the unit together. Pull trephine portion straight and evenly off the unit and place both pieces on procedural field with the flat side down. Discard the protective ring surrounding the trephine and the packaging band.

MARKING THE DONOR CORNEA

In order to mark the four quadrants on the epithelial side of the donor cornea, place the tip of a sterile surgical marking pen (gentian violet) into the four holes located in the well portion. Evenly coat each hole by twisting the pen.

ATTAINING SUCTION

DEPRESS the syringe plunger completely and hold. Ideally this is with the aid of a surgical assistant.
PLACE donor cornea in the well portion with the endothelial side facing up.
CENTER the tissue with fine tooth forceps.
INSERT the four posts of the seating ring into the holes of the well portion and gently press down. Abruptly release the syringe plunger. Pull the seating ring off the well and discard it.

CUTTING THE DONOR CORNEA

Next, insert the four pins of the trephine portion into the four holes located in the corners of the well portion.
IMPORTANT: Press down firmly on the flat surface of the punch and hold for 5 to 10 seconds to ensure trephine blade cuts completely through the donor tissue.

REMOVAL OF DONOR CORNEA

Pull trephine portion straight and evenly off the tissue. Using a fine tooth forceps, remove the scleral rim. The vacuum is released and the Donor button will remain in the well portion.
 Leaving the button undisturbed in the well until needed allows maximum contact with the ink, enhancing the epithelial marks. A few drops of the corneal storage medium on the button prevents desiccation.

	Lot number		Product is classified as a Medical Device
	Sterilized using gamma		Product Code
	Do not use if packaging is damaged		Unique Device Identifier
	Do not reuse		Use by date
	Consult instructions for Use <small>www.Labtician.com/docs</small>		Do not resterilize
	Caution		Keep Dry
	Single sterile barrier system		Prescription Only

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