## 4 REASONS WHY LABAMNIO™ MAKES MORE SENSE FOR YOUR PATIENTS AND YOUR PRACTICE

### CONFIDENCE

Proprietary **DryTek™** processing preserves the Amniotic Membrane's inherent properties for **tissue regeneration and healing**.

### CONVENIENCE

Can be stored at **room temperature** for **3 years**. Applied **straight off the pack**. Comes with **orientation markers** to guide amnion placement.

### COMFORT


Easily applied with a Bandage Contact Lens, with **no sutures** or **foreign body sensation**, and maximum comfort.

### CHOICE

Wide selection of shapes, sizes and thicknesses to ensure the right fit — **all at the same affordable cost**.

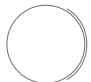| PRODUCTS              | LAB AMNIO 3x3<br>Amniotak™   | LAB AMNIO 12v<br>Amniotak™-Vision   | LAB AMNIO 12<br>Amniotak™-C   | LAB AMNIO 9<br>Amniotak™-C  | LAB AMNIO 9D<br>Amniotak™-C2  | LAB AMNIO 3x3D<br>Amniotak™-2  | LAB AMNIO G<br>Amniotak™-G   |
|-----------------------|--|---|---|---|---|--|--|
|                       | Each <b>LabAmnio™</b> product comes packaged in a <b>ReadyPack™</b> kit with a smooth-tip reusable forcep and a box of 10 PVA eye spears   |   |   |   |   |  |  |
| INDICATIONS           | <b>CONJUNCTIVAL SURFACE:</b> Pterygium surgery, chemical burns, conjunctival chalasis, leaking bleb<br><b>CORNEAL SURFACE RECONSTRUCTION:</b> Non-healing PED neurotrophic ulcers, keratitis sicca, overlay post PRK |   |   |   |   |  | Glaucoma Shunt Tube Covering   |
| DIMENSIONS            | 3x3cm<br>50-micron tissue  | 12 mm in diameter with 2 mm pin-hole<br>50-micron tissue  | 12 mm in diameter<br>50-micron tissue   | 9 mm in diameter<br>50-micron tissue  | 9 mm in diameter double thickness<br>100-micron tissue  | 3x3cm double thickness<br>100 - micron tissue  | 2x2cm triple thickness<br>150 - micron tissue  |
| ORIENTATION           | If the membrane's cut edge is facing right and away from the surgeon, the stromal side is facing down and the epithelial side is facing up.  | If the larger projection is distanced clockwise from the smaller one, the epithelial side is facing the cornea. | If the larger projection is distanced clockwise from the smaller one, the epithelial side is facing the cornea. | If the larger projection is distanced clockwise from the smaller one, the epithelial side is facing the cornea. | Double-layered graft. It can be placed in any orientation, as both sides are epithelial surfaces. | Double-layered graft. It can be placed in any orientation, as both sides are stromal surfaces. | Triple-layered graft. Both sides are stromal surfaces.                                     |
|                       |   |                                |                                |                                |                  |             |         |
| PRINCIPLES OF SURGERY | <b>INLAY OR GRAFT technique</b><br><b>OVERLAY OR PATCH technique</b><br><b>FILLING-IN OR LAYERED technique</b>   |   |   |   |   |  | LABAMNIO-G is specifically prepared to cover <b>GDD</b> (glaucoma drainage devices) tubes. |
| SHELF LIFE            | 3 YEARS  |   |   |   |   |  |  |
| STORAGE CONDITION     | Store in a clean, dry environment at <b>ROOM TEMPERATURE</b>   |   |   |   |   |  |  |
| ADVERSE EFFECTS       | The incidence of post-AM transplant microbial infections is very low with <b>DRY AMNIOTIC MEMBRANE</b>   |   |   |   |   |  |  |
| CONTRA-INDICATIONS    | Areas with active or latent infection   AM is suitable for transplantation only from an unopened, undamaged package<br>AM is intended for <b>SINGLE-USE ONLY I DO NOT RE-STERILIZE</b>                               |   |   |   |   |  |  |

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